


## INSTRUCTIONS FOR USE OF DISPOSABLE NEEDLE

# FAST-CUT®

The needle has been designed to be used with the Authomatic gun for biopsy

by STERYLAB ref. **FAST-GUN®**

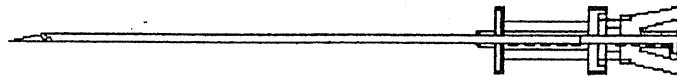

- WARNING:**
- Store in a dry and clean place away from heat sources
  - Before use ensure that the packaging is intact:  
if perforated, damaged or open discard the instrument
  - Check expiry date
  - Pay attention to not compromise the sterility of the instrument before use
  - In the event of a product defect or malfunction complete the yellow claim form to be found in the product carton and conserve the defective instrument

**N.B.:**

The use of this instrument is reserved to qualified medical personnel aware of its application.  
The instrument is sterile (sterilized by ETO GAS), single use and must be discarded after use

**Sterylab declines any responsibility in the event of improper use, reutilisation, resterilization.**

Fast-Cut can also be used also with the authomatic guns by BARD mod. "BIOPTY" and by MANAN mod. "PRO-MAG".

### PROCEDURE

- 1) Remove the protective cover of the needle. Check the integrity of the mandrel tip and, withdrawing the mandrel, the integrity of the cannula tip: if the latter catches the step of the distal side of the mandrel, discard the needle (light friction between the mandrel and the cannula is normal).
- 2) Load the needle into the automatic device with the "Sterylab" inscription facing down, following the instructions given by the manufacturer of the gun.
- 3) Use the automatic device according to the instructions given by the manufacturer of the gun.