

5mm-10mm laparoscopes

- |                        |                       |
|------------------------|-----------------------|
| I. Base Lever          | VI. Pin               |
| II. Positioning Arm    | VII. Locating Feature |
| III. Quick-Mount™ Lock | VIII. Quick-Mount™    |
| IV. Laparostat™        | IX. Exacting Lever    |
| V. Clamp               | X. Rail Adaptor       |

**INTENDED USE**


- The Laparostat holds a laparoscope and attaches into positioning arm.

**WARNING**

- *In the U.S.A., federal law restricts device to sale or use by or on order of a physician.*
- *For illustration purposes only, positioning arm and laparostat may be shown without a cover. Always place a cover over device to protect patients and users from cross-contamination.*
- *Do not use if device appears damaged.*
- *Positioning Arm Only: Forces exceeding 20lbs. (9.07kg) may move positioning arm when fully locked.*
- *Do not lubricate device.*

**Attaching Positioning Arm and Laparostat**



1. Attach positioning arm by turning base handle fully into attachment point.
2. Rotate lever to a desired position. Push base handle lever down to secure positioning arm.
3. Loosen Quick-Mount lock on positioning arm.
4. Insert Laparostat into Quick-Mount, ensuring pin is engaged into locating feature.
5. Tighten Quick-Mount lock on positioning arm to secure.
6. Place cover over positioning arm and Laparostat, using proper sterile technique (See *positioning arm cover instruction*).
7. Orient positioning arm to desired position. Use exacting lever to tighten arm.
8. For initial starting position, center telescoping section of Laparostat and position clamp.

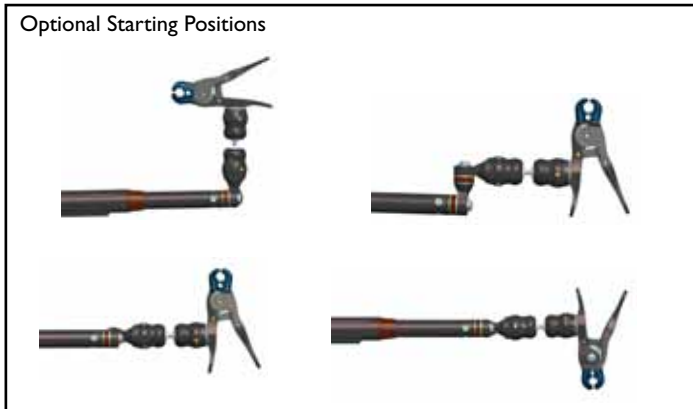**NOTE**

- *Reposition laparoscope by grasping the Laparostat at the clamp.*
9. Attach clamp to laparoscope as close to camera body as possible, taking care to avoid puncturing cover. Ensure laparoscope is secure.

**Cleaning and Disinfecting****WARNING**

- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*
  - *Do not place in mechanical washer.*
  - *Do not autoclave or immerse.*
  - *Sterility of positioning arm and Laparostat can only be achieved by using sterile positioning arm cover.*
1. After each use, discard single-use, disposable components.
  2. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

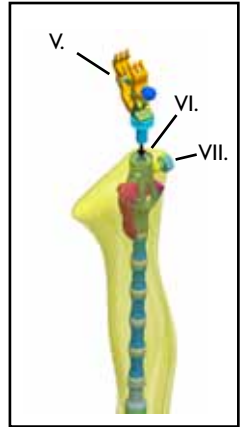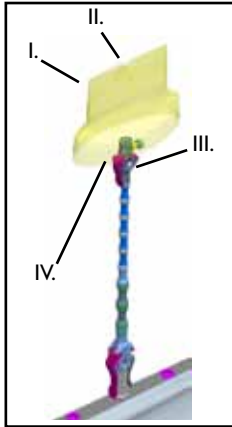

- I. Positioning Arm Cover
- II. Quick-Mount™ attachment opening
- III. Positioning Arm
- IV. Exacting Lever
- V. Attachment
- VI. Quick-Mount
- VII. Quick-Mount Lock