


MAMMOREP LOOP[®] MLX version

USE INSTRUCTIONS

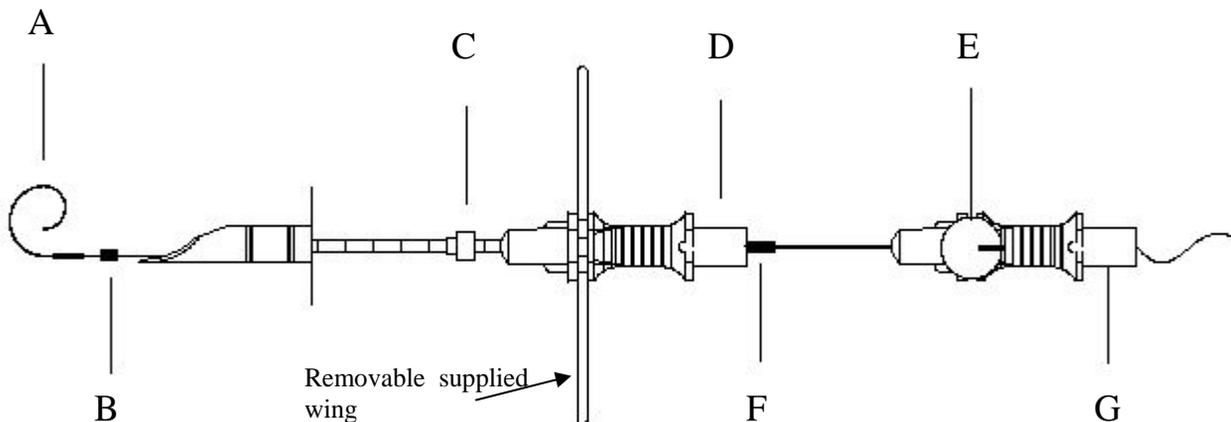

WARNING:

- Store in a dry clean place away from heat sources
- Before use ensure that the packaging is intact: if perforated, damaged or open discard the instrument
- Check the expiry date and that the model corresponds
- Peel open the package
- Pay attention to not compromise the sterility of the instrument before use
- In the event of a product defect or malfunction complete the yellow claim form to be found in the product carton and conserve the defective instrument

N.B.:

The use of this instrument is reserved to qualified medical personnel who are aware of its application.
The instrument is sterile (sterilized by ETO GAS) single use and should be discarded after use (Circ.Min:Sanità 403/9.5/1146-L. 10.81)

SeeDOS declines any and all responsibility in the event of improper use or reutilisation.

- 1) Remove the plastic tube leaving the sliding stopper **C** on the needle
- 2) Retract loop **A** into the needle, pulling grip **G** until mark **F** appears
- 3) Insert the MAMMOREP needle into the parenchyma up to the desired depth using centimetre marks and the depth stopper **C**
- 4) Once reached the lesion push out loop **A** by pushing grip **G** against grip **D**.
- 5) If new placement is needed pull grip **G** until mark **F** appears, to retract the loop **A**, and repeat the procedure from point 3)
- 6) Turn the screw **E** anticlockwise **twice or more** and extract the needle leaving the wire into the breast
- 7) Bend the wire out of the breast and secure it to the skin with a plaster
- 8) **The bead marker B warns surgeon that lesion is near**

**THE MAMMOREP HOOKWIRES ARE AVAILABLE TOGETHER WITH THE CONTAINER/LOCALIZER
FOR BIOOPTICALLY EXCISED IMPALPABLE NODULES**

X-SCREEN
(request sample)