


INSTRUCTION FOR USE OF CYTOLOGICAL ASPIRATION GUN **PAC/ PAC-UF**

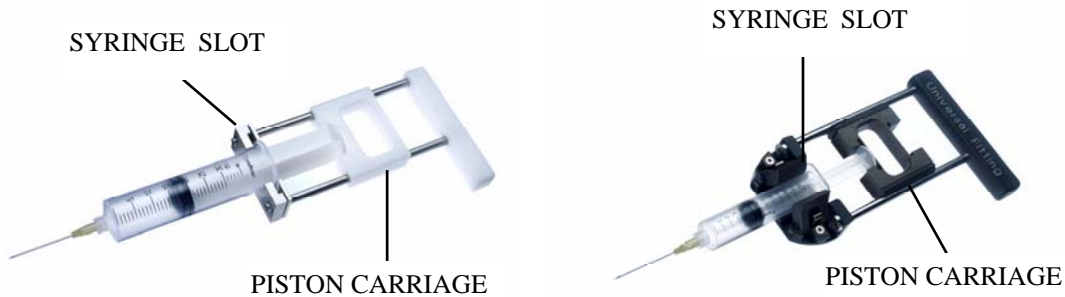

N.B.: - The instrument must be used by qualified medical personnel aware of its application.

- This instruction use don't indicate a medical or surgical technique. The doctor is responsible of the technique and the procedure implemented, using this instrument

SeeDOS Ltd declines any and all responsibility in the event of improper use.

USE PROCEDURE

- 1) Place the piston carriage in the right position to insert the syringe and the piston in the appropriate slot (only for PAC-UF: rotate the piston carriage and choose the right dimension of the syringe).
- 2) Insert the syringe and the piston in the appropriate slot.
- 3) Pull the piston carriage to perform the aspiration.

N.B. : it is recommended to execute disinfection/sterilisation procedure after each patient

STERILIZATION/DISINFECTION PROCEDURE

The biopsy gun can be sterilised either by steam or EtO gas, and disinfected by fluid immersion. Suggested sterilization parameters:

Steam: 130°C for 10-25 min. or 120°C for 15-30 min

EtO gas: 50°C, -70 kPa 16 hours cycle, de-gas cycle at 18°C for 72 hours

Disinfection is possible by fluid immersion using CIDEX®, PRO CIDEX® NS, following the procedure suggested by the manufacturer.

The User is responsible of the correct validation of the sterilisation cycle, both Steam or EtO gas.