

SeeDOS Product Users Manual



IMRT Virtual Water Phantom

REF 91235

Distributed by SeeDOS Ltd
For further information or a quotation
please contact cwalters@seedos.com

DOC #80364-00



General Precautions



WARNING:
Follow manufacturer's recommended safety procedures for radioactive sources.

Warnings and Cautions alert users to dangerous conditions that can occur if instructions in the manual are not obeyed. Warnings are conditions that can cause injury to the operator, while Cautions can cause damage to the equipment.



CAUTION:
Do not drop or mishandle unit.



CAUTION:
Refer all servicing to qualified individuals.




CAUTION:
Proper use of this device on careful reading of all instructions and labels.

Overview

The IMRT Virtual Water Phantom is designed to provide feedback on complex treatment plans generated using 3D. This Phantom is especially applicable to IMRT and tomotherapy modalities. The phantom is designed to mimic a human torso. Multiple thicknesses allow measurements for different body sizes. Both film and ion chambers can be simultaneously irradiated for comparisons with the treatment planning system.

Table of Contents

PAGE	
2	General Precautions
2	Overview
3	General Operation
6	Recommended Exradin Ionization Chambers
6	Maintenance
6	Service
7	Parts and Accessories List
7	Customer Responsibility
7	Bibliography
8	Features and Specifications
12	Warranty

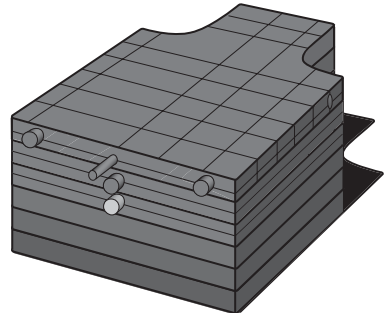

SeeDOS



General Operation

The IMRT Virtual Water Phantom is easily used for dose verification by the following steps:

1. Select appropriate phantom configuration to resemble the clinical patient setup desired.

Each 3 cm virtual water slice can be added for a total thickness range of 3 to 18 cm. Two pieces are solid and are placed in position where no data is to be collected. Two of the pieces have a cavity resembling the shape of a lung and are to be used when a large tissue inhomogeneity may present a complication in a treatment. The final two pieces contain a number of cavities for an ionization chamber. The ion chamber can be positioned anywhere along the cavity. Unused cavities are filled with the solid acrylic rods. For commonly treated cases such as prostate, it is best to use a specified location for the chamber center as indicated by the scribe line. Place the lung inserts (set of four) into the simulated lung cavities and the bone equivalent plug into one of the ion chamber cavities, if desired.

2. Scan the Virtual Water Phantom setup with CT.

Arrange the assembled IMRT Virtual Water Phantom on the CT table with the ion chamber in position. Scan as much of the phantom as possible ensuring that the entire length of the active area of the chamber is included. Export the scanned phantom to the treatment planning system and proceed to contour the active area of the chamber.

3. Position the phantom on the treatment unit.

Position the assembled IMRT Virtual Water Phantom on the treatment table placing isocenter in the middle of the desired chamber volume. The semitransparent acrylic is ideal for localization as the surface SSD may be easily read and at the same time the exact location of the chamber may be viewed with respect to the localization lasers. The scribe marks on the side of the phantom can be used for rapid isocenter placement. Place a piece of ready pack V type film at the appropriate level(s) where a coronal dose distribution is to be measured.

General Operation

4. Import the fluence pattern from the original treatment plan on the computer.

For most planning systems that use dynamic multileaf collimators this is a standard feature and will copy the exact fluence pattern from one field to another. After each fluence pattern is copied onto the phantom plan, calculate the dose. If dose to a particular point off the central axis is desired, then a separate contour will be needed and can be placed inside one of the cavities. Next, place fields around the phantom in the same manner they were used in the patient plan. This involves selecting the fields with the same gantry, collimator and jaw settings.

Dose distributions on the phantom can be measured by placing a piece of ready pack verification film between the acrylic slabs. If a high degree of correlation is desired, a small steel ball may be inserted in one of the cavities to provide a point of reference. The absolute dose to the chamber center may be calculated using the dose volume histogram utility available on most planning systems. If no such utility is available, the average dose may be computed by simply measuring the point dose on the planning system with a number of points and taking the average. If the dose volume histogram does not show a steep gradient at the expected dose level, then the chamber is either too big for the given field or in a region where the dose is changing rapidly and should not be used for an absolute measurement. If the exact location of the steel spheres is desired, make certain the film is pre-irradiated using an open field that covers the steel localization spheres embedded in the phantom. Pre-irradiation is acceptable since the film will only be used to show relative dose, not absolute dose. If one desires to have no irradiation in the area of the film to be

analyzed, then simply pre-irradiate the film using two exposures with the jaws collimated up such that only a thin strip approximately 1 cm wide is used to irradiate the section with the steel spheres.

5. Irradiate the phantom

Irradiate the phantom delivering the dose just as one would if a patient were on the treatment table. Deliver approximately 50 cGy to the chamber. This will allow simultaneous irradiation of the chamber and film with the same beams. As the charge is accumulating on the chamber for each field, record the amount of charge accrued for each beam if possible. This is useful if a significant deviation is found between the measured and expected dose.

6. Compare results from Phantom dosimetry with the original plan.

Using the appropriate calibration factors, convert the accumulated charge into an effective dose.

General Operation

Example (see TG 21 for definitions of terms): Using data measured for a given field at a depth of 10 cm:

$$\begin{array}{ll} M = 2.13 & N_{\text{gas}} = 4.663 \times 10^9 \\ nC = \text{Measured} & L/r = 1.063 \\ \text{charge for a given} & P_{T,P} = 1.022 \\ \text{monitor setting} & P_{\text{ion}} = 1.000 \\ P_{\text{repl}} = 1.000 & \end{array}$$

Use the equation:

Dose in water (D_w) =

$$M * N_{\text{gas}} * L/r * P_{T,P} * P_{\text{repl}} * P_{\text{ion}}$$

Dose in water (D_w) =

$$2.13 * 4.663 * 1.063 * 1.022 * 1.000 * 1.000$$

$D_w = 11.16$ cGy where P_{phan} is given in Table 1 for 6X at 10 cm deep.

Develop the film and scan using film dosimetry software. The film may indicate a maximum dose that is higher or lower than what is given on the plan by up to 20% based on variations of film, temperature, processor conditions, etc. This deviation should not present a problem since the relative dose distribution on film compared with the treatment plan is of interest. Once the film is digitized, compare the location of selected isodose lines with that from the planning system. Any significant deviation should be investigated.

7. Other measurements

Point doses can be measured off axis using the optional TLD/Diode/MOSFET Phantom, REF 70608. If a point dose such as spinal cord is needed this can be accomplished by inserting the TLD/Diode/MOSFET Phantom in the approximate position above or below the desired isocenter. Once the exact location is

determined from the patient plan, a small lead marker may be placed into one of the multiple slots and taped in place. The lead marker will appear when the phantom is scanned by the CT and the image imported into the planning system. A diode dosimeter or TLD may be placed in the location and irradiated along with the film and chamber. This will give an accurate point dose in a high gradient region without affecting the overall measurement.

Recommended Exradin Ionization Chambers

General Chamber Specifications

For informational purposes, the specifications for most chambers useful for this application are given below.

Humidity: 10-80%, non-condensing

Temperature: -15 to 50° C

Nominal Collection Efficiency: 100%

Maximum Polarizing Potential: Greater than 1000 V

Normal Polarizing Potential: 300V

The Exradin chamber models given in the table below are normally used for IMRT applications. The specifications for each chamber are listed below.

Nominal Leakage Currents: 10⁻¹⁵ A

Cable: 50 Ω, 29 pF/ft, 2 m long

Connector: Triaxial BNC Plug, 2-Lug Male (shell of chamber is common with connector body)

Model	A1	A1SL	A14	A14SL	A16	A12	A12S
Collecting Volume (cm ³)	0.056	0.056	0.009	0.009	0.007	0.65	0.25
Nominal Calibration Factor (R/nC)	60	60	365	365	400	5	14
Centroid of Collecting Volume (mm from tip of chamber)	4.0	4.1	2.0	2.1	1.0	12.9	5.8
Collector Diameter (mm)	1.0	1.0	1.5	1.5	0.3	1.0	1.0
Outside Diameter of Shell Collecting Volume (mm)	6.0	6.25	6.0	6.25	3.4	7.1	7.1
Wall Thickness (mm)	1.0	1.1	1.0	1.1	0.5	0.5	0.5
Shell, Collector, & Guard Material	C552	C552	C552	C552	C552	C552	C552

Model Specifications

Maintenance

Exterior cleaning of the device can be done with a soft brush and a cloth. Gently brush all surfaces to remove dirt and dust. Remove any remaining dirt with a cloth slightly dampened with a solution of mild detergent and water or a liquid disinfecting agent.

Service

There are no serviceable parts on the IMRT Virtual Water Phantom.

Parts and Accessories List

REF	Description
72183	IMRT Virtual Water Phantom Diode Slab
50086	Additional Virtual Water Phantom Slab
30728	Additional Solid Virtual Water Plugs
Custom	Custom cavity drilled in 30728 for any Ion Chamber
72175	Exradin A1/A14 plug
72176	Exradin A1SL/A14SL plug
72174	Exradin A12 plug
72178	Phantom PR-06 Farmer Chamber
70614	PTW 3000 series plug
92705	A1 Exradin Miniature Shonka Thimble Chamber, 0.056 cc
92722	A1 Exradin Slim Line Miniature Shonka Thimble Chamber, 0.056 cc
92711	A14 Exradin MicroChamber, 0.009cc
92723	A14 Exradin Slim Line MicroChamber, 0.009cc
92700	A12 Exradin Waterproof Farmer Type Chamber, 0.6 cc

Please contact SeeDOS for a variety of Custom Plugs available

Customer Responsibility



Federal law in the U.S.A. and Canada restricts the sale, distribution or use of this device to, by or on the order of a licensed medical practitioner. The use of this device should be restricted to the supervision of a qualified medical physicist. Handling of radioactive sources is potentially hazardous and should be performed by qualified personnel. Should repair or replacement of this device become necessary after the warranty period, the customer should seek advice from Standard Imaging

Inc. prior to such repair or replacement. If this device is in need of repair, it should not be used until all repairs have been made and the product is functioning properly and ready for use. The owner of this device has sole responsibility for any malfunction resulting from abuse, improper use or maintenance, or repair by anyone other than Standard Imaging Inc. The information in this manual is subject to change without notice. No part of this manual may be copied or reproduced in any form, or by any means, without prior written consent of Standard Imaging Inc.

Bibliography

1. "A protocol for the determination of absorbed dose from high energy photon and electron beam." TG21 Med. Phys 10:(6) 741-771 (1983)

Features and Specifications

Dimensions:

Height (six slabs)	7.09 in. (18.00 cm)
Width (each slab)	11.81 in. (30.00 cm)
Length (each slab)	17.72 in. (45.00 cm)

Weight (six slabs) 50.0 lbs. (22.7 kg)

Individual Components:

- (2) Chamber Virtual Water Phantom Slab with 6 cavities for Ion Chamber placement
- (2) Blank Phantom Slabs for build up thickness
- (2) Lung Phantom Slabs with cavities for simulated lungs
- (16) Virtual Water Plugs to fill unused Ion Chamber cavities
- (1) Virtual Water Plug with cavity for Chamber of your choice
- (1) Bone Equivalent Plug
- (1) Lung equivalent insert set to fill lung phantom voids

Notes

Notes

Warranty

Standard Imaging, Inc. sells this product under the warranty herein set forth. The warranty is extended only to the buyer purchasing the product directly from Standard Imaging, Inc. or as a new product from an authorized dealer or distributor of Standard Imaging, Inc.

For a period of twenty-four (24) months for well chambers and twelve (12) months for all other Standard Imaging, Inc. products from the date of original delivery to the purchaser or a distributor, this product is warranted against functional defects in materials and workmanship, provided it is properly operated under conditions of normal use, and that repairs and replacements are made in accordance herewith. The foregoing warranty shall not apply if the product has been disassembled, altered or repaired other than by Standard Imaging, Inc. or if the product has been subject to abuse, misuse, negligence or accident.

Standard Imaging's sole and exclusive obligation and the purchaser's sole and exclusive remedy under the above warranties are limited to repairing or replacing free of charge, at Standard Imaging's option, a product: (1) which contains a defect covered by the above warranties; (2) which are reported to Standard Imaging, Inc. not later than seven (7) days after the expiration date of the 12 or 24 month warranty period; (3) which are returned to Standard Imaging promptly after discovery of the defect; and (4) which are found to be defective upon Standard Imaging's examination. Transportation charges are the buyer's responsibility. This warranty extends to every part of the product except fuses, batteries, or glass breakage. Standard Imaging, Inc. shall not be otherwise liable for any damages, including but not limited to, incidental damages, consequential damages, or special damages. Repaired or replaced products are warranted for the balance of the original warranty period, or at least 90 days.

This warranty is in lieu of all other warranties, express or implied, whether statutory or otherwise, including any implied warranty of fitness for a particular purpose. In no event shall Standard Imaging, Inc. be liable for any incidental or consequential damages resulting from the use, misuse or abuse of the product or caused by any defect, failure or malfunction of the product, whether a claim of such damages is based upon the warranty, contract, negligence, or otherwise.

This warranty represents the current standard warranty of Standard Imaging, Inc. Please refer to the labeling or instruction manual of your Standard Imaging, Inc. product for any warranty conditions unique to the product.

Distributed by SeeDOS Ltd

For further information or a quotation please contact

Colin Walters at cwalters@seedos.com

Tel : +44 (0) 1525 850670

Fax: +44 (0) 1525 850685